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## Instruction for use csp ceramic silane primer

**1. Product description**  
Mixable two-component primer for the silanization of silicate or aluminum oxide ceramics. To improve the bond strength with resin-based materials or to improve the bond strength between composite cement and fiberglass-reinforced composite posts.

**2. Composition**  
**csp base:** gamma-methacryloxypropyltrimethoxysilane  
**csp activator:** water, acetic acid

**3. Intended Use**  
Ceramic bonding and auxiliary bonding promoters are intended for the indirect reconstruction or correction of functionally compromised natural dentition (e.g. deficient teeth).

**4. Indication**  
Ceramic bonding for a permanent chemical bond between silicate or aluminum oxide ceramics, fiberglass-reinforced composite root posts and composite materials.

**5. Contra-indication**  
Known allergy to methacrylates.

**6. Patient target group:**  
SAREMCO *csp ceramic silane primer* is suitable for use in all patients without any age or gender restrictions.

**7. User**  
SAREMCO *csp ceramic silane primer* should only be used by a professionally trained dental practitioner.

**8. Side effects**  
None known.

**9. Interactions**  
None known.

**10. Processing stages**

**10.1. Preparation**  
If necessary, roughen the surfaces to be treated, dry and keep dry.

**10.2. Mixing**  
Mix **two (2)** drops of *csp base* with **one (1)** drop of *csp activator* on a mixing plate. Wait 5 minutes until the liquid appears homogeneous and transparent.

**Caution:** the mixed liquid loses its effectiveness after about 8 hours.

**10.3. Application**  
Apply the mixed liquid with a brush and massage in until a shiny film is formed. Gently blow dry with a gentle stream of air. Keep dry!

**10.4. Further processing**  
Proceed with the protocol immediately. Conditioning and application of the adhesive (SAREMCO recommends cmf bond, but without prior application of cmf prime!) and the composite (SAREMCO recommends els cem, apt composite and els composite) according to the instructions for use of the product used.

**11. Storage**  
Close *csp* bottles tightly after use. Store in an upright position to ensure the backflow of the liquid. *csp* was developed for use at room temperature (20°C - 25°C / 68°F - 77°F). Store at temperatures between 4°C - 28°C / 39°F - 82°F. If room temperature exceeds 28°C / 82°F storing in the refrigerator is recommended. Do not freeze! Bring to room temperature before use.

**12. Batch number and expiry date**  
The batch number should be specified to identify products in the case of enquiries. Products should no longer be used once the expiry date has elapsed.

**13. Precautionary measures**  
Close containers after each use with the right lid. *csp activator* is corrosive! Avoid contact with skin, oral mucosa, eyes, and clothes. Keep out of reach of children. For dental use only. The principles of good personal and safe working practice apply. Commercially available medical gloves do not provide protection against the sensitization effect of methacrylates. If the product comes in contact with the glove, remove the glove and dispose of it, wash your hands with water and soap immediately and put on a new glove. In case of an allergic reaction, seek medical advice.

**14. Emergency measures**  
In case of direct contact with skin, oral mucosa or eyes, rinse immediately and thoroughly under running water. In case of contact with the eyes, consult an eye specialist.

**15. Hygiene**  
Use application instruments for one patient only. Dose products away from patients to avoid contamination.

**16. Warranty**  
The product was developed for use in dentistry and must be processed in accordance with the instructions for use. For further damages, namely that caused by non-compliance with the instructions for use or other improper handling or inappropriate use of a product, any liability is rejected. Our liability is restricted to the quality of our products. In the case

of a product being of defective quality, only its value is replaced. It is the responsibility of the user to check, before using the products, whether they are suitable for the intended purpose. He expressly assumes all risks associated with using the product and is solely responsible for any resulting damages. Safety data sheets and technical data sheets are available on the website of SAREMCO Dental.

**17. Other notes for Europe**  
If the user and/or patient becomes aware of serious incidents in connection with the use of the product, they are to be reported to the manufacturer and the responsible authorities of the state in which the user and/or patient resides. The Summary of Safety and Clinical Performance of SAREMCO products can be found in the European database on medical devices (EUDAMED-<https://ec.europa.eu/tools/eu-damed>).

**18. Production / distribution**  
SAREMCO Dental AG  
Gewerbestrasse 4  
CH-9445 Rebstein / Switzerland  
Tel: +41 (0) 71 775 80 90  
Fax: +41 (0) 71 775 80 99  
info@saremco.ch  
www.saremco.ch

Edited 07-2024 | D600053

Class III medical devices  
Class III medical devices (Canada)

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