Gebruichsanweisung csp ceramic silane primer

1. Produktbeschreibung

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Instruction for use csp ceramic silane primer

1. Product description

Mixable two-component primer for the silanization of silicate or aluminum oxide ceramics. To improve the bond strength with resin-based materials or to improve the bond strength between composite cement and fiberglass-reinforced composite posts.

2. Composition

csp base: gamma-methacryloxypropyltri-methoxysilane
csp activator: water, acetic acid

3. Indication

Ceramic bonding for a permanent chemical bond between silicate or aluminum oxide ceramics, fiberglass-reinforced composite root posts and composite materials.

4. Contra-indication

Known allergy to methacrylates.

5. Side effects

None known.

6. Interactions

None known.

7. Processing stages

7.1. Preparation

If necessary, roughen the surfaces to be treated, dry and keep dry.

7.2. Mixing

Mix two (2) drops of csp base with one (1) drop of csp activator on a mixing plate. Wait 5 minutes until the liquid appears homogeneous and transparent. **Caution:** the mixed liquid loses its effectiveness after about 8 hours.

7.3. Application

Apply the mixed liquid with a brush and massage in until a shiny film is formed. Gently blow dry with a gentle stream of air. Keep dry!

7.4. Further processing

Proceed with the protocol immediately. Conditioning and application of the adhesive (SAREMCO recommends cmf bond, but without prior application of cmf prime!) and the composite (SAREMCO recommends apt composite and els composite) according to the instructions for use of the product used.

8. Storage

Close csp bottles tightly after use. Store in an upright position to ensure the backflow of the liquid. csp was developed for use at room temperature (20°C - 25°C / 68°F - 77°F). Store at temperatures between 4°C - 28°C / 39°F - 82°F. If room temperature exceeds 28°C / 82°F storing in the refrigerator is recommended. Do not freeze! Bring to room temperature before use.

9. Batch number and expiry date

The batch number should be specified to identify products in the case of enquiries. Products should no longer be used once the expiry date has elapsed.

10. Precautionary measures

Close containers after each use with the right lid, csp activator is corrosive! Avoid contact with skin, oral mucosa, eyes and clothes. Keep out of reach of children. The principles of good personal and safe working practice apply. Commercially available medical gloves do not provide protection against the sensitization effect of methacrylates. If the product comes in contact with the glove, remove the glove and dispose of it, wash your hands with water and soap immediately and put on a new glove. In case of an allergic reaction, seek medical advice.

11. Emergency measures

In case of direct contact with skin, oral mucosa or eyes, rinse immediately and thoroughly under running water. In case of contact with the eyes, consult an eye specialist.

12. Hygiene

Use application instruments for one patient only. Dose products away from patients to avoid contamination.

13. Warranty

Our liability is restricted to the quality of our products. In case of a product being defective quality, only its value is replaced. For further damages, namely that caused by non-compliance with the instructions for use or other improper handling or inappropriate use of a product, any liability is rejected. It is the responsibility of the user to check, before using the products, whether they are suitable for the intended purpose. He expressly assumes all risks associated with using the product and is solely responsible for any resulting damages. Safety data sheets and technical data sheets are available on the website of SAREMCO Dental.

14. Production / distribution

SAREMCO Dental AG
Gewerbstrasse 4
CH-9445 Rebstein / Switzerland
Tel: +41 (0) 71 775 80 90
Fax: +41 (0) 71 775 80 99
info@saremco.ch
www.saremco.ch

Edited 10/2020 | D600053

Class IIa medical devices
Class III medical devices (Canada)

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