Gebrauchsanweisung els unibond

1. Produktbeschreibung

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Instruction for use els unibond

1. Product description

Light-curing, 1-component self-etching adhesive to create a permanent marginal-gap-free adhesion between the tooth structure (dentine, enamel) and the light-curing filling/fixing material.

2. Composition

ethanol, water, BisEMA, methacrylated phosphoric salt, silica, polyalkenoate methacrylated, initiators

3. Indication

1. Direct restorations with light-curing composite-based materials.
2. Indirect restorations: at the use of light-curing composite cements to fix inlays, onlays, crowns and bridges.

4. Contra-indication

Opened pulp, pulpititis, known allergy to methacrylates.

5. Side effects

In individual cases, contact allergies have been described to products of a similar composition. To avoid pulp reactions, it is recommended to cover the exposed dentine in the pulp area with suitable underfilling materials (e.g. with calcium hydroxide). Note: els unibond does not contain HEMA, TEGDMA or BisGMA.

6. Interactions

Avoid underfilling materials which may hinder polymerization owing to their ingredients. All phenolic compounds, such as zinc oxide eugenol or preparations containing thymol, belong to this category.

7. Processing stages

7.1. Drying

Due to the following application of the filling material, a drainage of the working field is required. Avoid any contamination of the cavity with blood or saliva. A rubber dam is recommended.

7.2. Cavity preparation

Prepare the cavity as usual. Undercuts and bevelled margins are recommended to improve adhesion and the margin fit. Clean and dry the cavity. Avoid overdryning.

7.2.1. Etching of tooth substance (optional)

By applying els unibond clinically sufficient bond strength values are achieved. By selective enamel etching or in total-etch technique even higher adhesion values can be achieved.

7.2.1.1. Total-etch technique (with SAREMCO cmf etch)

Apply the etching gel and condition in correspondence to instruction for use of the product. SAREMCO recommends cmf etch.

7.2.1.2. Selective enamel etching

Apply the etching gel condition in correspondence to instruction for use of the product. SAREMCO recommends cmf etch. Sufficiently etched enamel appears matt and white.

Note: Possibly use the total-etch-technique in cavities limited by dentine. In this case the dentine should not be etched for more than 15 seconds and only dried briefly after rinsing.

7.2.2. Non-Etch

Do not overdry the dentine. A slightly moist cavity surface is optimal. Without preparation (e.g. cervical defect), thoroughly clean the surface and roughen the enamel margin with a finishing diamond.

7.3. Application

Apply els unibond to enamel/dentine in a not too thin layer with a brush and massage into the tooth substance for 20 seconds. Note: els unibond is light-curing, so avoid too intense ambient light. Dim the surgical light during the application.

7.4. Drying

Dry the adhesive layer with an air blower for at least 5 seconds.

7.5. Polymerizing

Polymerize the bonding layer for 20 seconds with blue light (halogen or LED light). When using dual-curing luting composites, light-curing of the luting composite is mandatory.

Note: Do not use els unibond for opaque restorations or deep cavities where complete light-curing is not guaranteed.

7.6. Notes for fixing Maryland bridges

When fixing Maryland bridges with els unibond, the enamel must be roughened or etched with phosphoric acid gel (e.g. SAREMCO cmf etch) prior to use of the bond. Self-etching bonds are less effective on non-prepared enamel. Excessive amount of bond material onto unpolished and unbrushed enamel can cause edge staining. For optimal bonding results with els unibond, the enamel should be prepared in advance in class III, IV, V and for diastema closures.

8. Storage

Closes els unibond bottle tightly after use. Store in an upright position to ensure the backflow of the liquid. Do not expose light-curing products to direct sunlight or operating light. els unibond was developed for use at room temperature (20°C - 25°C / 68°F - 77°F). Store at temperatures between 4°C - 28°C / 39°F - 82°F. If room temperature exceeds 28°C / 82°F storing in the refrigerator is recommended. Do not freeze! Constant temperatures above 28°C / 82°F can reduce the shelf-life of the product.

9. Batch number and expiry date

The batch number should be specified to identify products in the case of enquiries. Products should no longer be used once the expiry date has elapsed.

10. Precautionary measures

Close containers after each use with the right lid. Keep out of reach of children. Commercially available medical gloves do not provide protection against the sensitisation effect of methacrylates. If the product comes into contact with the glove, remove the glove and dispose of it, wash your hands with water and soap immediately and put on a new glove. In case of an allergic reaction, seek medical advice.

11. Emergency measures

In case of direct contact with the oral mucosa, rinse with water. In case of contact with the eyes, rinse thoroughly with water. Consult an eye specialist.

12. Hygiene

Use application instruments for one patient only. Dispose products away from patients to avoid contamination.

13. Warranty

Our liability is restricted to the quality of our products. In case of a product being of defective quality, only its value is replaced. For further damages, namely that caused by non-compliance with the instructions for use or other improper handling or inappropriate use of a product, any liability is rejected. It is the responsibility of the user to check, before using the products, whether they are suitable for the intended purpose. He expressly assumes all risks associated with using the product and is solely responsible for any resulting damages. Safety data sheets and technical data sheets are available on the website of SAREMCO Dental.

14. Other notes for Europe

If the user and/or patient become aware of serious incidents connected with application of the product, they are to be reported to the manufacturer and the responsible authorities of the state in which the user and/or patient resides.

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